Medisanté BC800 3G

Body Composition Scale

Intelligente Körperanalyse-Waage
Balance connectée d'analyse corporelle
Scala di composizione del corpo

Instruction Manual

Bedienungsanleitung
Manuel d'utilisation
Manuale d'uso

M+edisanté

Medisanté BC800 3G

Summary

EN Instruction Manual

Package contents
Display Indicators
Operating Instructions
Important Safety Information
Specification

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
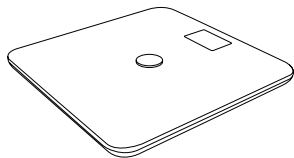
Read the instruction manual before using the device

Intended use: The Medisanté Body Composition Scale BC800 3G is a digital monitor for home/ domestic use which measures weight and body composition values of persons 18 years of age or more.

Package contents

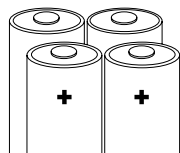
Medisanté Body Composition Scale
BC800 3G


4 AAA
Alkaline batteries

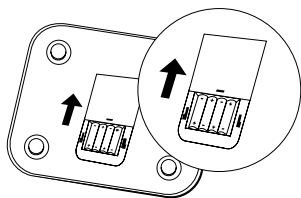1. Insert batteries

- 01** Place the scale upside down on a flat surface being careful not to scratch it.
- 02** Remove the battery cover and insert 4 AAA alkaline batteries with correct polarity.

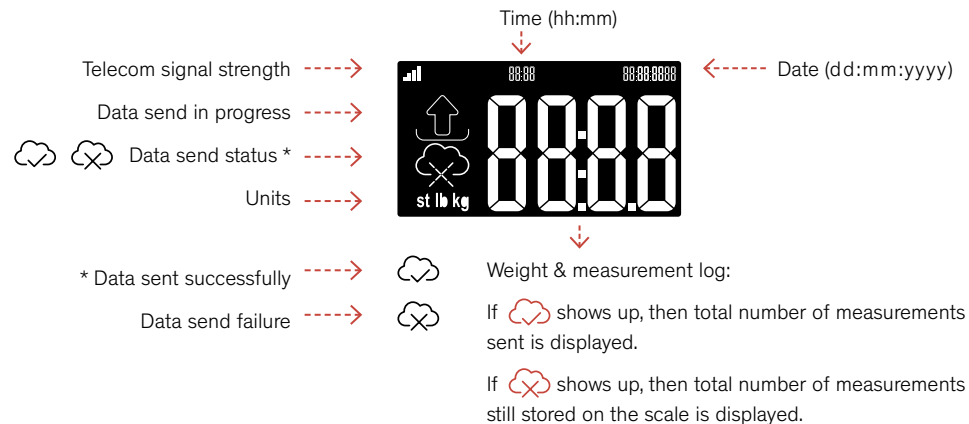Display Indicators

Overload Indicator

ERR

Scale is overloaded (max. 180 kg, 396 lbs, 28.3 st)

Low Battery Indicator

LO

Batteries are low and need to be replaced.

- 01** Remove battery cover.
- 02** Insert batteries.
- 03** Replace battery cover.

2. Operating Instructions

- 01** Place the scale on a hard, flat surface.
- 02** Step on the scale and remain still during measurement.
- 03** When the “data send in progress” indicator is displayed, the data is being sent and you may step off the scale.
- 04** After a period of time, the data send status will be displayed. If it shows a “data send failure”, move the scale to a new location and try again.

Notes

- *Weight measurements are possible with shoes and socks on. Body composition values are only possible with barefoot measurements.*
- *Calibration: If you place the Body Composition Scale BC800 3G at a new place, quickly step with one foot on it, until the display turns on. Step off and wait until the calibration gets performed (10 – 15 seconds) and the display switches off.*
- *If there is no signal strength indicator and you continue to receive a data send failure, contact your distribution partner.*

Automatic Shutdown

If no measurement is being taken, the scale will automatically turn off in 10 seconds.

Important Safety Information

- Please read the entire instruction manual and safety precautions before using the Body Composition Scale BC800 3G.
- The body composition percentages (muscle, water, fat) are only indicative. For further information, please consult a health care professional or dietitian.
- Unusual measurement values should be discussed with your doctor or health care professional.
- Never change the dosage of any prescribed medications without first consulting your doctor or health care professional.
- Always perform the measurements on a hard and flat surface. Uneven floors may affect the readings.
- If necessary, clean the Body Composition Scale BC800 3G with a damp cloth and a small amount of dishwashing liquid.
- Never immerse the scale in water or rinse it under running water.
- Protect the Body Composition Scale BC800 3G against knocks, moisture, dust, chemicals, drastic changes in temperature and electro-magnetic fields.
- Keep the Body Composition Scale BC800 3G away from any sources of heat.
- Never step on the outer edge of the glass. This could cause the scale to tip or damage the glass.
- Keep packaging materials away from children.
- Store the Body Composition Scale BC800 3G horizontally, not vertically.
- If the measurement appears inaccurate or if the LCD display does not show any figures, please check that the batteries are correctly inserted and the battery contacts are clean.
- If the Body Composition Scale BC800 3G will not be used for an extended period of time, remove the batteries from the Body Composition Scale BC800 3G and store it properly. Do not store other objects on top of it.

To Avoid Injury

- Only use this device for its intended use and in accordance with the instruction manual.
- Do not use the device if it is damaged in any way.

Specification

Sensors	High precision strain gauge sensors
Maximum Capacity	180 kg / 396 lbs / 28.3 st
Increments	0.1 kg / 0.2 lbs / 0.01 st
Measurements	Weight and body impedance to calculate body composition percentages (muscle, water, fat)
Display	White backlight LCD display
Fault indications	Overload and low battery
Data transmission	Over a 3G/2G mobile connection (with Vodafone SIM card inside)
Operating environment	Temperature 10~40°C (50~104°F) Humidity 15 %~90 % RH
Frequency bands	GSM/GPRS/EDGE: Dual band GSM 900/1800MHz UMTS/HSPA+: Dual band UMTS 900/2100MHz
Output power	Class 4 (+33dBm $\pm$ 2dB) for EGSM900 Class 1 (+30dBm $\pm$ 2dB) for GSM1800 Class E2 (+27dBm $\pm$ 3dB) for GSM 900 8-PSK Class E2 (+26dBm +3 /-4dB) for GSM 1800 8-PSK Class 3 (+24dBm +1/-3dB) for UMTS 2100, WCDMA FDD Bdl Class 3 (+24dBm +1/-3dB) for UMTS 900, WCDMA FDD BdVIII
Automatic shutdown	Yes, after 10 seconds

Hereby, EBM Technologies Inc. declares that the radio equipment type BC800 3G is in compliance with Directive 2014/53/EU.

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© 2017 Medisanté. All rights reserved.
EBM Technologies Inc. dichiara che l'apparecchio modello BC800 3G è conforme alla Direttiva 2014/53/UE.
EvivaMed GmbH
EvivaMed Schweiz GmbH
c/o BDO AG
Medisanté AG